



EYE VISC™ PFS

User -Friendly &
Ready to Use

**HYDROXYPROPYL METHYLCELLULOSE
OPHTHALMIC SOLUTION USP 2% w/v**

- Provides a better control during procedure as it is available with polycarbonate plunger.

- Crystal clear solution of highly purified grade of HPMC
- Excellent coating ability
- Better cell and tissue protection
- Excellent flow property

Indications: Anterior segment surgery, Routine cataract surgery, Phaco-emulsification IOL implantation, Corneal Transplantation, Glaucoma surgery

Also Available

EYE VISC™ Plus PFS

HYDROXYPROPYL METHYLCELLULOSE OPTHALMIC SOLUTION USP 2% w/v

The plus advantage with a HIGHER viscosity
(Viscosity; 10,000 cPs)

Available in 2ml PFS packing



EYE VISC™ PFS

HYDROXYPROPYL METHYLCELLULOSE OPHTHALMIC SOLUTION USP

CE
0434

Composition

Hydroxypropyl Methylcellulose USP.....2% w/v
Sterile Isotonic base.....q.s

Description:

EYE-VISC™ PFS is a sterile non-pyrogenic, viscoelastic preparation of a non-inflammatory, highly purified grade Hydroxypropyl methylcellulose 2% w/v dissolved in a physiological buffer at pH 6.0 - 7.8 for intra-ocular application during the anterior segment surgeries.

Composition:

Each ml of EYE-VISC™ PFS contains 20 mg Hydroxypropyl methylcellulose USP in sterile isotonic vehicle.

Characteristics:

Highly retentive, dispersive viscoelastic. Unique ability to offer good space maintenance and excellent tissue protection throughout the procedure.

Possesses a high dynamic viscoelasticity and coating ability. Characterized by a low molecular weight and low surface tension. Totally non antigenic and completely transparent. Does not require refrigeration.

Indications:

For intra-ocular use only. EYE-VISC™ PFS is indicated whenever protection and lubrication on delicate cells and tissues are needed especially in ophthalmic procedures including : Anterior Segment Surgery Cataract Surgery and IOL implantation Corneal Transplantation & Glaucoma Surgery

Contraindications:

Not known at present

Warning and Precautions:

The cannula and syringe are for single intra-ocular use only. Check the integrity of the packaging before use to ensure the product has remained sterile.

The Intra-ocular pressure should be checked following operations and if significant increase is observed, appropriate therapy should be administered.

At the end of operations, the solution should be removed by irrigation / aspiration.

Supply:

EYE-VISC™ PFS is packed and supplied as a sterile solution in 2ml pre-filled syringe with cannula.

Warning:

To be sold by retail on the prescription of a Registered Medical Practitioner only.

- Store at room temperature. • Protect from light. - Contains no preservative. • Must not be reused & re-sterilized. • For intra-ocular use only • **Single Use Only**

Dose:

As directed by the Physician



For further informations, please write to :

Bio-Tech Ophthalmics Pvt. Ltd.

First Floor, Plot No. 555-556-557, Khatraj Vadsar Road, Khatraj, Tal. Kaloj, Dist. Gandhinagar. (Gujarat) INDIA • Ph. +91 2764 281 336/317
E-mail : intbusiness@biotechvisioncare.com (Export) • sales@biotechophthalmics.com (Domestic)

Website: www.biotechvisioncare.com

For the use of a Registered Medical Practitioner or a Hospital or a Laboratory